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Translation of the original user's manual

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CE-Declaration of conformity

The company AAT Alber Antriebstechnik GmbH herewith declares that the escalino is conform with the current regulations of the European Union, EEC Guidelines 2006/42/EG, 2007/47/EG and 93/42/EG. This declaration becomes void if the escalino have been altered without the consent of AAT Alber Antriebstechnik GmbH.

Signature of the manufacturer:

Markus Alber
(Managing shareholder)

The stair climber escalino has been successfully tested according to the standards ISO 7176-23 and DIN EN 12182 at the Technical University of Berlin, Berlin-Cert.

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Table of content

1	Symbols and safety instructions07
2	Description of the product.10
2.1	Features of the device.10
2.2	Technical data13
2.3	Dimensioned sketch15
2.4	Scope of delivery16
2.5	Accessory.16
3	Assembly17
3.1	Back rest17
3.2	C-part plus seat18
3.3	Helix cable19
3.4	Arm rests20
3.5	Disassembly20
4	Initial operation21
4.1	Adjust handles21
4.2	Turning on the device22
4.3	Neutral position23
4.4	Checking the safety brakes24
4.5	Single step mode25
4.6	Check capacity of the batteries25
5	Operation27
5.1	Person taking a seat.27
5.2	Drive on level ground30
5.3	Drive on slopes31
5.4	Climbing upstairs34
5.5	Climbing - downstairs36
5.6	Laying down on the stairs38
5.7	Parking38
5.8	Transportation.40

6	Care and maintenance42
6.1	Batteries42
6.2	Exchange the electrical fuses46
6.3	Check the lining of the brakes and the climbing feet.46
6.4	Cleaning/Desinfection48
6.5	Recycling49
7	Malfunction and trouble shooting50
7.1	Malfunction50
7.2	Trouble shooting53
8	Warranty and liability54
8.1	Warranty54
8.2	Liability55

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1 Symbols and safety instructions

WARNING - Can lead to serious injuries or death!

CAUTION - Can lead to light injuries!

NOTE - Can lead to material damage!

- » Before initial operation, read the user's manual and take note of the warning and safety instructions.
- » Before initial operation, check the electronic functions, the safety brakes and the lining of the climbing feet.
- » Before initial operation, store the escalino in non-freezing environment.
- » Take note of the technical data.
- » The approved total weight and the max. weight of the person may not be exceeded.
- » Operation only by one trained person, by a minor of 16 and above only under supervision. This person should be physically and intellectually capable to handle the escalino safely.
- » Always operate with both hands.
- » Practice driving without a person and with single step mode.
- » Wear non-slip footwear.
- » For the transportation of persons, not of loads.
- » Secure persons who can not seat on their own with a belt.

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① Handle (adjustable in height)

② UP/DOWN switch

③ Safety chip for the engine immobiliser

④ Back rest (removable)

⑤ Arm rest (swivels out/removable)

⑥ C-part plus seat

⑦ Safety brake

⑧ Main wheel

⑨ Climber and climbing foot

⑩ Helix cable

⑪ Climbing unit

⑫ Membrane keypad

⑬ Clamp lever for back rest

⑭ Clamp lever for height adjustment grip

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2.2 Technical data

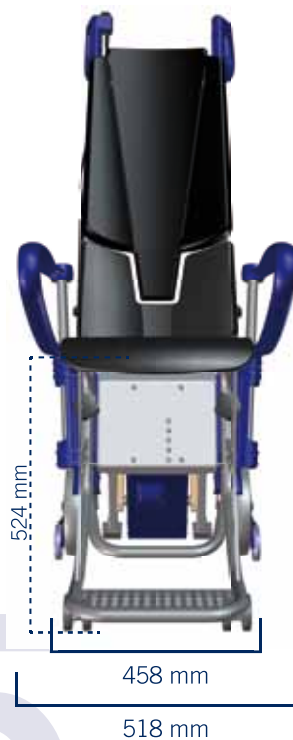
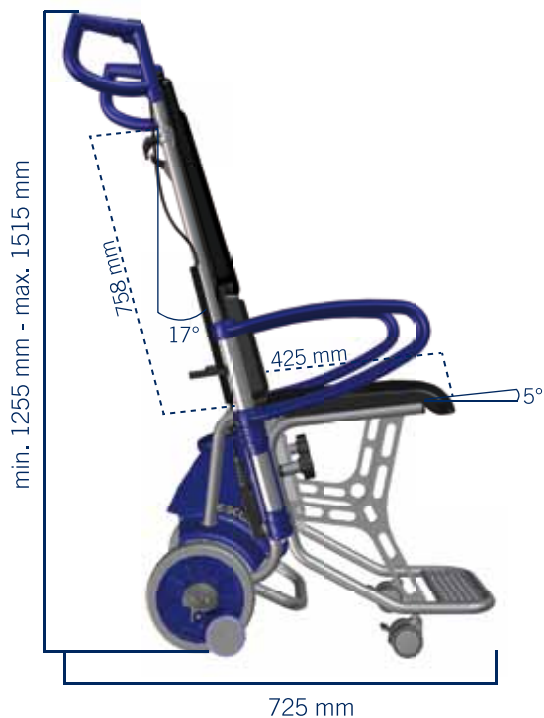
Technology	escalino
Measurements	Height = min. 1255 - max. 1515 mm Width without arm rests = 458 mm Width with arm rests = 518 mm Depth = 725 mm
Weight	Climbing unit = 22,0 kg Back rest = 7,5 kg C-part plus seat = 3,8 kg Both arm rests = 1,5 kg Charger = 0,2 kg Total weight = 35,0 kg
Total weight approved	155 kg
Max. weight of the person	120 kg
Climbing speed	approx. 14 steps / minute
Range with one charge of the batteries on average load of 75 kg	approx. 30 levels (equates 450 steps)
Range with one charge of the batteries on maximum load	approx. 13 levels (equates 200 steps)
Max. height of step	210 mm
Min. depth of the step's surface	120 mm
Necessary step width (straight stairs)	Width escalino + 100 mm
Min. platform size (U-shaped stairs)	Depends on the attendant's height
Batteries	2 x 12 V / 5 Ah (sealed lead acid)

Direct current motor	24 V / 200 W
Noise level	The device's A weighted noise level lies typically below 70 dB (A)
Storing/operating temperatur: device	0° C up to +60° C / -30° C up to +50° C
Storing/operating temperatur: batteries	0° C up to +60° C / -30° C up to +50° C
Vibrations	The weighted RMS value of acceleration the upper body is exposed to lies typically below 2,5 m/s ² .
Sticker serial number	On the left side of the housing.

In the interest of progress we reserve the right for technical changes.

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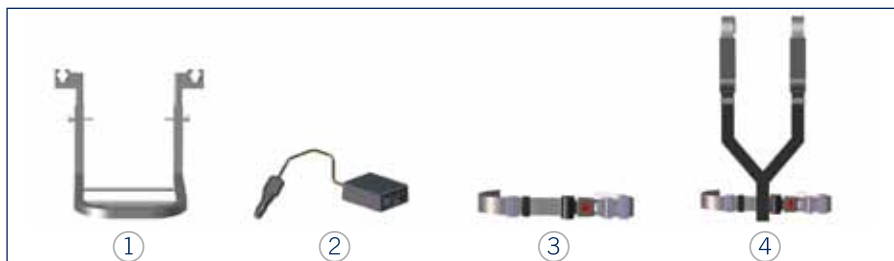
Secure with clamp levers.
» Note the position of the clamp levers (chapter 3.4)!

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WARNING - Tighten the
tightening knobs!

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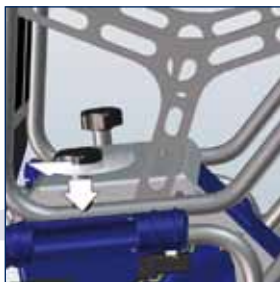
Slide the c-part plus seat.
» Bring the bolt into the
nut!

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Latch in drill and tighten
the tightening knobs.

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Put helix cable into its
socket and lock it.

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3.4 Arm rests

Pull the clamp lever outwards and change position.

- » Clamp lever shows in the direction of the operator!

Insert on both sides completely.

- » For a wider or narrower seat width, switch the arm rests.

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NOTE - Before initial operation, store escalino in non-freezing environment!

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Adjust handles.
» Shoulder height recommended!

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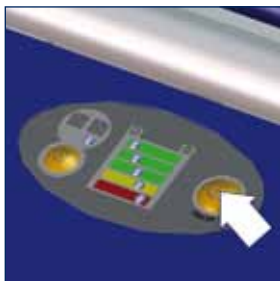
4.2 Turning on the device

Insert safety chip for the engine immobiliser.

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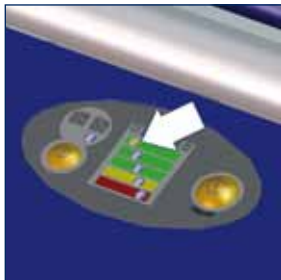
4.3 Neutral position

Bring climbers by pressing the UP/DOWN switch into neutral position.

» If single step mode is activated, climbers will stop automatically in neutral position.

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WARNING - Do not use the escalino, if the required brake effect (on one side or both sides) is not guaranteed! Contact manufacturer or authorized personnel!

WARNING - Check safety brakes without a person!

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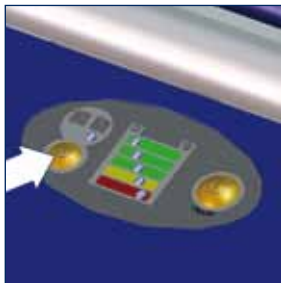
Test: escalino cannot be pushed forward, only backward.

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
4.5 Single step mode

Deactivate / Activate.

- » Single step mode is automatically activated after turning on the escalator!
- » LED glows = single step mode deactivated!

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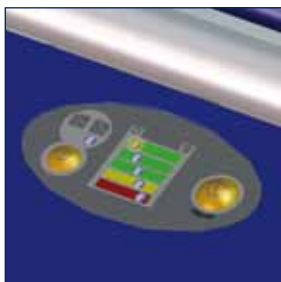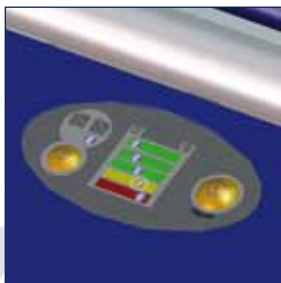
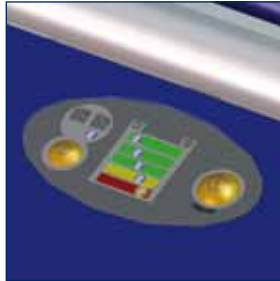
Red = empty

- » Charge batteries!
- » Audio warning every 12 sec.!
- » The capacity of the batteries is sufficient for max. 20-35 steps!

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WARNING - Operation only by one trained person!

WARNING - Practice driving without a person and with single step mode!

WARNING - Always operate with both hands!

CAUTION - The arms should not protrude over the arm rests!

NOTE - Wait a few seconds after turning on the escalino before you push the UP/DOWN switch!

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Lift the arm rests and turn them back.

Platz nehmen.

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Take a seat.

- » If necessary secure the person with a belt!

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- » Evtl. Armlehnen tauschen (Kapitel 3.4)!

Lift the arm rests and turn them in front.

- » Switch the arm rests optionally (chapter 3.4)!

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5.2 Drive on level ground

Loosen the locking
of the steering rollers'.

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Drive.
» Overcome thresholds
backwards!

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Lenkrollen

5.3 Drive on slopes

Loosen the locking
of the steering rollers'.

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Tilt the escalino and keep
it in balance.

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**Drive downhill**

Forward.

- » To slow down, tilt the escalino a bit more!

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Rückwärts.

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**Drive uphill**

Backwards.

- » To slow down, tilt the escalino a bit more!

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WARNING - Keep the escalino always under tension!

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Tilt the escalino and keep it in balance (chapter 5.3, picture 3-5).

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Drive backwards to the step all the way.

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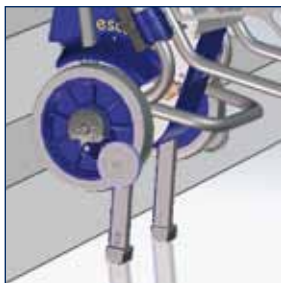
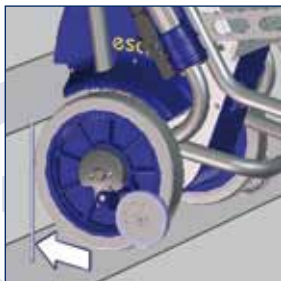
WARNING - As soon as the climbers resume load, you feel tension in the handles. Hold escalino lower briefly to counteract this!

WARNING - Further persons are not allowed in the danger area underneath the escalino!

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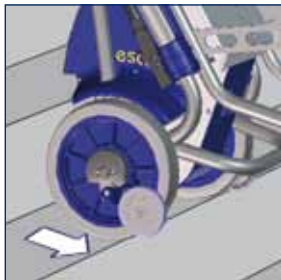
Tilt escalino and keep it in balance (chapter 5.3, picture 3-5).

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Push escalino forward until safety brakes grip.

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Press UP/DOWN switch direction **DOWN**.

» If single step mode is activated, climbers will stop automatically in neutral position.

escalino steigt hinab.

escalino descends.

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Push the escalino forward.
» Repeat from picture 4.

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5.6 Laying down on the stairs

Lay down the escalino.

- » Safety brakes prevent from rolling down!

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NOTE - Charge the batteries after each use!

The automatic switch off turns the escalino **OFF**, during longer brakes of non-use. Turn the escalino **OFF** first and then **ON**.

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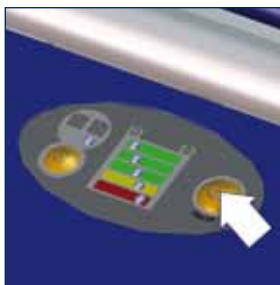
Remove the safety chip for
the engine immobiliser.

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WARNING - Take note of the weight specifications!

WARNING - During transportation in vehicles, secure the escalino according to the relevant legal regulations!

CAUTION - Turn the escalino off for transportation!

The batteries are not considered dangerous according to the IATA special guidelines A67 and GGVS Rn-Nr. 2801 A, Abs. 2!
Batteries are admitted to air travel by DOT and IATA!
Disassemble the escalino (chapter 3.5)

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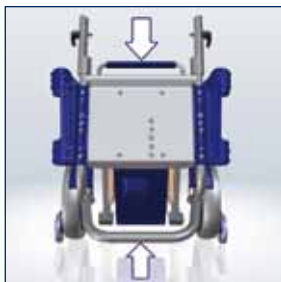
Lift the climbing unit on the upper and lower frame.

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WARNING - We recommend a safety check every two years!
Tip: We recommend a safety check, if the escalino goes to a new user.

WARNING - Repairs and safety checks may only be done by the manufacturer or authorized personnel!

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CAUTION - Never touch with moist hands!

NOTE - Protect the escalino against humidity!

NOTE - Charge the batteries after each use!

NOTE - Protect the charger and the voltage converter from oils, grease, aggressive cleaning agents, thinners or damage!

NOTE - Follow the user's manual of the charger and the voltage converter!

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NOTE - A total discharge lead to a loss of the batteries' capacity and shortens their durability!

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New batteries reach their full capacity after several chargings.

Batteries are admitted to air travel by DOT and IATA.

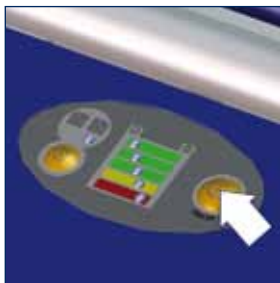
The batteries are maintenance free and rechargeable.

Akku laden

escalino ausschalten.

Charge batteries

Turn off the escalino.

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Connect charger.

» Keep the plug-in contacts clean!

D

GB

LED grün = Akku voll

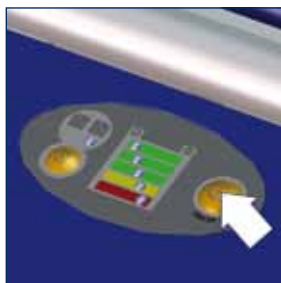
LED green = batteries full

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
LED orange = batteries are being charged

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
Charge batteries in a vehicle
Turn off the escalino.

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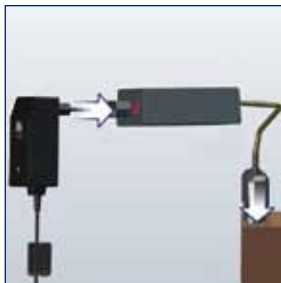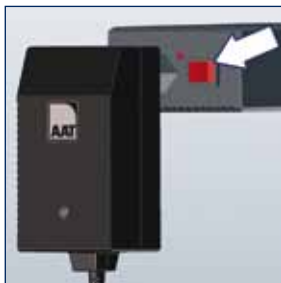Turn on the voltage
converter.
» Audio warning = voltage
low!

LED grün = Akku voll

LED green = batteries full

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LED orange = batteries are
being charged

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NOTE - Only by the manufacturer or authorized personnel!

6.3 Brems-/Steigfußbeläge prüfen

6.3 Check the lining of the brakes and the climbing feet

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NOTE - Minimum thickness = 1 mm. Should the lining be thinner or damaged, contact the manufacturer or the authorized personnel!

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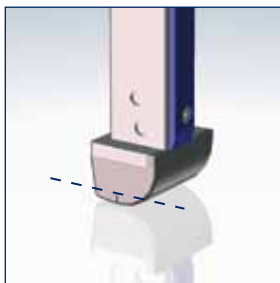
WARNING - Check safety brakes after cleaning (chapter 4.4)!

WARNING - Do not smoke during the cleaning!

NOTE - Clean the safety brakes, the brake drums in the wheel rims weekly or after contamination with spirit. Follow the hazard notes of the spirit!

NOTE - Keep the escalino away from ignition source!

NOTE - Do not use a high-pressure cleaning device!

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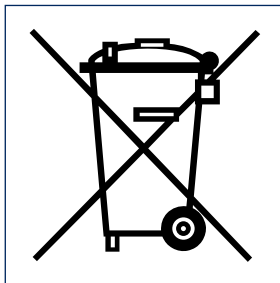
With a moist cloth and household cleaners / disinfectants.

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6.5 Recycling

The escalino and its batteries are products of long durability. However, if your escalino is old and has reached the end of its lifespan you may return its components to AAT Alber Antriebstechnik GmbH or the authorized personnel to be recycled.

A new law concerning electric and electronic devices (ElektroG) has been passed on March 24, 2005 regulating the taking back and recycling of old electric and electronic devices.

According to the Federal Ministry for the Environment (German BMU) the escalino is considered a means of transportation and does not fall under this law concerning the recycling of old electric or electronic devices (ElektroG, §2 paragraph 1 sentence 1 (cross reference, Guideline of the European Parliament 2002/96/EG)).

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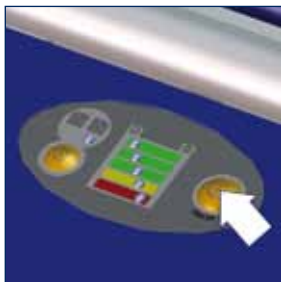
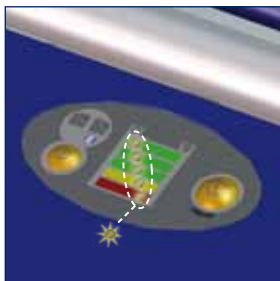
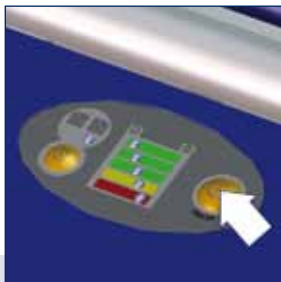
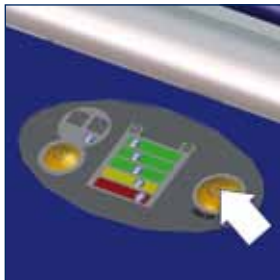
Turn on the escalino.

- » LED is blinking - Contact the manufacturer or the authorized personnel!

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Malfunction

LED blinking	Malfunction
1x	socket is defective
2x	error, circuit board (Fet)
3x	error, circuit board (relay)
4x	UP/DOWN switch is defective
5x	error motor

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7.2 Trouble shooting

problem	check and potential solution
escalino cannot be turned on	Is the helix cable put in and locked? No: Put helix cable into its socket and lock it.
	Is the charger connected to the escalino? Yes: Disconnect the charging cable.
	Are the batteries empty? Yes: Charge the batteries.
	Is the problem still present? Yes: Contact the manufacturer or the authorized personnel.
escalino does not drive	Is the safety chip for engine immobiliser inserted? Yes: Contact the manufacturer or the authorized personnel.
Batteries don't charge	Contact the manufacturer or the authorized personnel.

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8 Warranty and liability

8.1 Warranty

AAT Alber Antriebstechnik GmbH grants a warranty of two years for the escalino including all its accessory parts (with the exception of the sealed lead acid batteries). The warranty begins at the date of purchase and covers failure of material and errors in manufacturing.

For the sealed lead acid batteries, always assuming correct maintenance, AAT Alber Antriebstechnik GmbH grants a warranty of one year from the date of purchase.

The warranty becomes void concerning defects or damage which are the result of the following:

- » damage due to excessive strain.
- » damage due to improper handling.
- » violent damage.
- » natural wear and tear (brake lining, gliding strip, climbers).
- » improper and insufficient maintenance.
- » unauthorized constructional changes or inappropriate use of the escalino or its accessories.
- » using the escalino exceeding the total weight approved or the max. weight of the person.
- » improper charging of the sealed lead acid batteries.

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8.2 Liability

The manufacturer AAT Alber Antriebstechnik GmbH, is neither liable nor responsible for potential damage of the escalino in case of:

- » improper handling of the escalino.
- » repairs, fittings or other works performed on the escalino by unauthorized personnel.
- » operation not in accordance with the user's manual.
- » extraneous parts added or connected to the escalino.
- » parts removed from the escalino.
- » operation exceeding the total weight approved or the max. weight of the person.
- » persons under age use the escalino (except for those by minor of 16 and above only under supervision).
- » operation of the escalino with insufficiently charged batteries.

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