D/GB

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Translation of the original user's manual

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CE-Declaration of conformity

The company AAT Alber Antriebstechnik GmbH herewith declares that the SOLO is conform to the current regulations of the European Union, EEC Guidelines 2006/42/EG, 2007/47/EG and 93/42/EG. This declaration becomes void, if the SOLO has been altered without the consent of AAT Alber Antriebstechnik GmbH.

Signature of the manufacturer:

Markus Alber
(Managing shareholder)

The wheel hub drive SOLO has been successfully tested according to the standards DIN EN 12182 and DIN EN 12184 at the Technical University of Berlin, Berlin-Cert.

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Table of contents

1	Symbols and safety instructions.	07
2	Description of the product	11
2.1	Features of the device	11
2.2	Technical data	18
2.3	Dimensioned sketch	20
2.4	Scope of delivery	21
2.5	Accessories	22
3	Assembly	23
3.1	Powered wheels	23
3.2	Wheels with quick-release axle	25
3.3	Battery pack	26
3.4	Control unit.	29
3.5	Anti-tilt support.	31
3.6	Disassembly	33
4	Initial operation	38
4.1	Engage the powered wheels	38
4.2	Turn on the SOLO	39
5	Operation	40
5.1	Display	42
5.2	Driving the SOLO	49
5.3	Driving manually with powered wheels	52
5.4	Driving with quick-release axle wheels	54
5.5	Transportation	55

6	Care and maintenance	57
6.1	Battery pack57
6.1.1	Charge lithium ion battery pack.59
6.1.2	Charge lead silicone battery pack.63
6.2	Cleaning/Desinfection64
6.3	Recycling.65
6.4	Safety checks66
6.5	New owner66
7	Warning notices, malfunction and troubleshooting	67
7.1	Warning notices67
7.2	Malfunction74
7.3	Troubleshooting77
8	Warranty and liability.	78
8.1	Warranty78
8.2	Liability.79

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1 Symbols and safety instructions

WARNING - Can lead to serious injuries or death!

CAUTION - Can lead to light injuries!

NOTE - Can lead to material damage!

- » Before initial operation, read the user's manual carefully and take note of the warning and safety instructions as well as the wheelchair's limiting values.
- » Before initial operation, check the electronic function, anti-tilt supports, brackets and, if applicable, the steering rollers' air pressure.
- » Before initial operation, store the SOLO at least one hour in a non-freezing environment.
- » Take note of the technical data.
- » The approved total weight may not be exceeded.
- » Due to the mounting of the SOLO the measurements of the wheelchair can change.
- » Operation only by one trained person. This person should be physically and intellectually capable to handle the SOLO safely.
- » Practice outside on level ground and at the lowest possible speed.
- » For the transportation of persons, not of loads.
- » Secure persons who can not seat on their own with a belt.
- » Never touch rotating or moving parts.

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③

Receptacle

Adapting shell for the wheels with quick-release axles

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2.2 Technical data

Technology	SOLO	
	Lithium-Ionen	lead silicone
Measurements	Battery pack: Height = 191 mm Width = 180 mm Depth = 111 mm	
Weight	Both powered wheels = 18.0 kg Bag for battery pack = 0.2 kg Control unit = 0.4 kg Interface = 0.1 kg	
	Battery pack = 4.2 kg	Battery pack = 5.8 kg
	Total weight (without wheelchair) = 22.9 kg	Total weight (without wheelchair) = 24.5 kg
Total weight approved	210 kg ⁽¹⁾ (person, wheelchair, SOLO)	
Max. weight of the person	160 kg	
Wheelchair enlargement due to attachment	Approx. 60 mm	
Possible wheel sizes	24 "	
Speed, forward backward	6 km/h 3 km/h	
Range with one charge of the batteries	up to 35 km ⁽²⁾	up to 12 km ⁽²⁾
Braking-/climbing capacity	up to 20% ⁽³⁾	
Batteries	28.8 V/16 Ah	24 V/8,5 Ah

Direct current motor	2x 28.8 V/150 W
Noise level	The device's A weighted noise level lies typically below 60 dB (A)
Storing/operating temperature: device	0° C up to +50° C / -10° C up to +50° C
Storing/operating temperature: battery pack	0° C up to +50° C / -10° C up to +50° C
Charging temperature: Lithium ion battery pack	0° C up to +35° C
Vibrations	The weighted RMS value of acceleration the upper body is exposed to lies typically below 2,5 m/s ² .
Class of application	A, B
Sticker serial number	interface

⁽¹⁾ Observe the maximum load according to the wheelchair manufacturer's stipulations.

⁽²⁾ According to ISO 7176-4, the range depends on the person's weight, the underground, the territory, the temperature, etc.

⁽³⁾ At an approved weight of 120 kg, the braking - / climbing capacity also depends on the underground, the territory, the temperature, etc.

In the interest of progress we reserve the right for technical changes.

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Antriebsrad positionieren.

Position the powered
wheel.

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Location bolt in receptacle
device.

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Lock the powered wheel.
» It must click into place!

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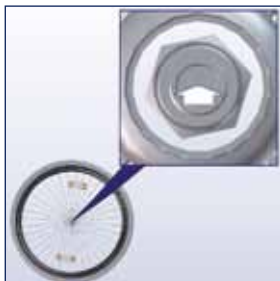
WARNING - The wheels with quick-release axles must lock!

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NOTE - Keep the plug-in contacts dry!

**Go into driving mode -
Lithium ion battery pack**
Loosen the screws.

Turn the switch rail and insert it.
» Pay attention to the arrows!

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**Connecting**

Insert into the bag and close the Velcro fastening.

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Plug in the interface.

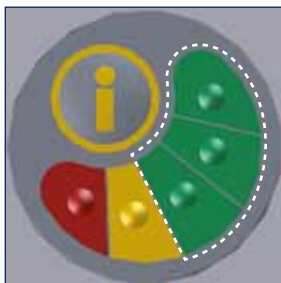
Akku-Pack prüfen

Infotaste am Akku-Pack drücken.

**Check battery pack**

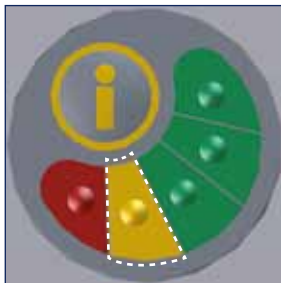
Press the information key on the battery pack.

Grün = voll

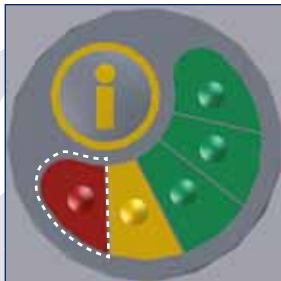
Green = full

Gelb = gering
» Ggf. laden!

Yellow = little charge
» Charge, if applicable!

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Attach the bracket for the
control unit.

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Put the control unit on the
adjustment part.

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WARNING - Before initial operation check the anti-tilt supports!

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Should there be no anti-tilt support on your wheelchair – use AAT anti-tilt support!

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Slide in anti-tilt support.
» Note identification “R” and “L”!

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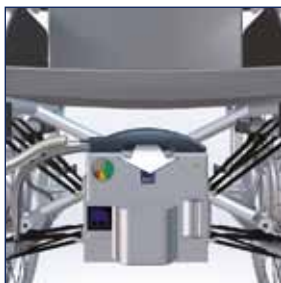
Unfasten Velcro fastener
and remove the battery
pack.

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**Wheels with quick-release
axles**

Press the unlocking knob
on the axle.

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WARNING - The wheelchair's locking brakes must be adjusted properly!

WARNING - Before initial operation, check electronic function, anti-tilt supports, brackets and, if applicable, the steering rollers' air pressure!

NOTE - Before initial operation, store the SOLO at least one hour in a non-freezing environment!

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Turn the powered wheels on the handrims until you hear them lock.

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WARNING - Operation only by one trained person! This person should be physically and intellectually capable to handle the SOLO safely!

WARNING - Practice outside on level ground and at the lowest possible speed!

WARNING - Never use the SOLO without anti-tilt supports!

WARNING - Use the lowest possible speed when driving on slopes!

WARNING - Always go backwards over curbs / obstacles!

WARNING - Do not drive parallel to steep mountain sites or tracks!

WARNING - Keep a safe distance to curbs!

WARNING - Do not drive too closely to stairs or precipices!

WARNING - When crossing streets, intersections and train tracks, use extra caution!

WARNING - Avoid using the SOLO in heavy rain, ice, snow, and on unsecured grounds (sand, mud, loose gravel)!

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WARNING - Avoid driving down slopes manually with powered wheels!

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WARNING - When not in use, e.g. while using mass transportation (bus, subway, etc.) or when you change seats, turn off the SOLO and engage the locking brakes on the wheelchair!

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WARNING - Do not use the SOLO on escalators, stairs or moving belts!

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WARNING - Driving on public roads, observe traffic regulations!

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NOTE - Do not drive backward to a wall with mounted anti-tilt supports!

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Warning, undervoltage,
battery pack

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WARNING - Never touch rotating or moving parts!

CAUTION - If the drive elements block or the driving characteristics change, turn off the SOLO immediately and stop using it!

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Note the chapter on warnings, malfunction and troubleshooting!

The display's light turns itself off after 20 seconds of non-use.

During longer brakes of non-use, the automatic switch-off turns the SOLO off after 10 minutes.

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Check battery capacity.

- » 5 bars = full
- » 3 bars = half full
- » 1 bar = empty -> charge!

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» 360° stufenlos!

Move the joystick in the desired direction.

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Press the keys simultaneously for 2 seconds.

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5.3 Driving manually with powered wheels

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CAUTION - Danger of jamming between wheels and side part / arm rest!

SOLO ausschalten.

Turn off the SOLO.

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Disengage powered wheel.

D

GB

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Disengage powered wheel.

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Drive manually.

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5.4 Driving with quick-release axle wheels

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CAUTION - Danger of jamming
between wheels and side part /
arm rest!

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Disassemble powered
wheels (see chapter 3.6).

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Assemble wheels with
quick-release axle (see
chapter 3.2).

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SOLO demontieren (s.h. Kapitel 3.6).

WARNING - Should the battery pack be damaged or defect, do not use it! Contact the manufacturer or authorized personnel!

WARNING - Do not use the SOLO plus wheelchair as a seat in a vehicle!

WARNING - During transportation in vehicles, secure the SOLO according to relevant legal regulations!

CAUTION - Turn off the SOLO for transportation!

CAUTION - Observe weight specifications!

NOTE - Transport the battery pack in its original packaging (keep the box)!

NOTE - Do not lay down the powered wheels on their handrims!

The battery pack is approved for air freight by DOT and IATA. However, inform airline ahead of time.

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Put into flight mode -
Lithium-Ionen battery pack
Loosen screws.

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Turn switch rail and insert
it.
» Pay attention to arrows!

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WARNING - We recommend a safety check every two years! Inspection plate informs about the latest date for next inspection!
Tip: We recommend a safety check, if the SOLO goes to a new user!

WARNING - Repairs and safety checks may only be done by the manufacturer or authorized personnel!

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WARNING - Never short-circuit or connect the battery pack's poles!

WARNING - Should the battery pack catch fire, put it out with sand only!

WARNING - Should the battery pack be damaged or defect, do not use it! Contact the manufacturer or authorized personnel!

CAUTION - When charging the SOLO, make sure the cable lies on safe ground!

NOTE - Protect the SOLO against humidity!

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NOTE - Never carry the charger at the charging cable!

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NOTE - Charge the battery pack after each use!

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NOTE - Protect the charger and the voltage converter for the car from oils, grease, aggressive cleaning agents, thinners or damage!

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NOTE - Observe the specifications on the charger!

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Battery pack is admitted to air travel by DOT and IATA (observe chapter 5.5).

The battery-pack is maintenance free and rechargeable.

It is not possible to use the SOLO while it is charging.

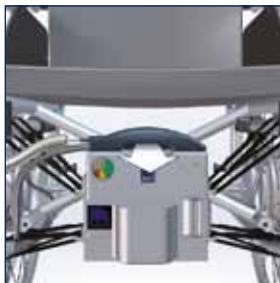
SOLO ausschalten.

Turn off the SOLO.

6.1.1 Lithium-Ionen Akku-Pack laden 6.1.1 Charge lithium ion battery pack

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Unplug interface, if applicable.

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Connect charger.

LED grün = Akku voll
» ca. 8 Stunden

LED green = batteries full
» approx. 8 h

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LED orange = batteries are being charged

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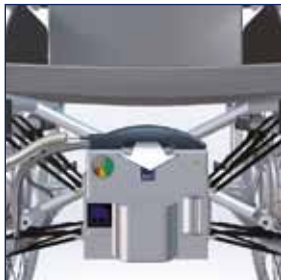
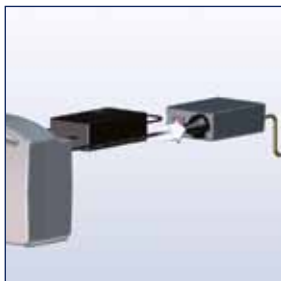
Unfasten Velcro fastener
and remove battery pack.

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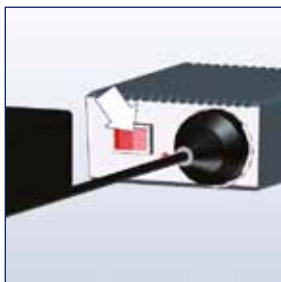
Connect the voltage
converter for the car and
the charger.

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» Warnton = Spannung
gering!

Turn on the voltage conver-
ter for the car.

» Acoustic signal = low
voltage!

LED grün = Akku voll
» ca. 8 Stunden

LED green = batteries full
» approx. 8 h

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LED orange = batteries are being charged

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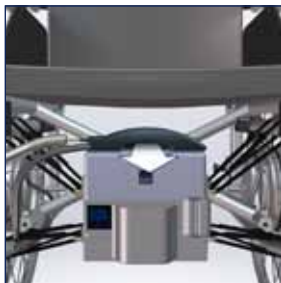
Check the capacity at the battery pack (see chapter 3.3)

6.1.2 Blei-Silikon Akku-Pack laden

6.1.2 Charge lead silicone battery pack

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Unplug interface, if applicable.

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Connect charger.

LED grün = Akku voll
» ca. 4,5 Stunden

LED green = batteries full
» approx. 4,5 h

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LED orange = batteries are being charged

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NOTE - Keep the SOLO away from ignition sources!

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NOTE - Clean the wheelchair according to the manufacturer's specifications!

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NOTE - Do not use a high-pressure cleaning device!

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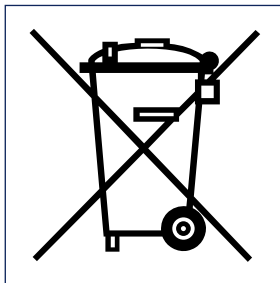
With a moist cloth and mild household cleaners / disinfectants.

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6.3 Recycling

The SOLO and its battery pack are products of long durability.

However, if your SOLO is old and has reached the end of its lifespan you may return its components to AAT Alber Antriebstechnik GmbH or authorized personnel to be recycled.

A new law concerning electric and electronic devices (ElektroG) has been passed on March 24, 2005 regulating the taking back and recycling of old electric and electronic devices.

According to the Federal Ministry for the Environment (German BMU) the SOLO is considered a means of transportation and does not fall under this law concerning the recycling of old electric or electronic devices (ElektroG, §2 paragraph 1 sentence 1 (cross reference, Guideline of the European Parliament 2002/96/EG)).

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WARNING - Safety checks may only be done by the manufacturer or authorized personnel!

WARNING - We recommend a safety check every two years! Inspection plate informs about the latest date for next inspection!
Tip: We recommend a safety check, if the SOLO goes to a new user!

A safety check is mandatory before the device changes hands. To conduct a safety check it is necessary to attend a training course. Please check the homepage for further information.

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6.5 New owner

The SOLO is a device suitable for multiple use. Before the SOLO changes hands it must be cleaned, disinfected and pass a safety check (see chapter 6.2).

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7 Warning notices, malfunction and troubleshooting

7.1 Warnhinweise

7.1 Warning notices

Radverriegelung

Beide Antriebsräder

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Wheel lock

Both powered wheels

- » Wheel and both arrows are blinking!

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Powered wheel on the left

- » Wheel and left arrow are blinking!

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Powered wheel on the right

- » Wheel and right arrow are blinking!

Antriebsräder richtig
(s.h. Kapitel 3.1)

adaptieren

Adapt powered wheels appropriately (see
chapter 3.1)

Kupplungsscheibe

Beide Antriebsräder

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**Coupling disc**

Both powered wheels

- » Hub and both arrows are blinking!

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Powered wheel on the left

- » Hub and left arrow are blinking!

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Powered wheel on the right

- » Hub and right arrow are blinking!

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If necessary, engage the wheels properly.

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Release the joystick and continue driving considering the reason for the message (remove blockade, turn around on slopes that are too steep).

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Stop driving the SOLO and let it cool.

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» Pfeil blinkt!

**Getting up function of the wheelchair**

» Arrow is blinking!

Rollstuhl in Sitz-/Normalposition bringen.

Bring the wheelchair's seat in its original position.

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7.2 Malfunction

Malfunction, battery pack

Code	Malfunction	Troubleshooting
01	No connection to the battery pack	Contact manufacturer!
02	Battery cell	Contact manufacturer!
03	I ² C error	Contact manufacturer!
04	Parameter data	Put into flight mode, wait 10 sec. Put into driving mode! Malfunction still present? Yes: Contact the manufacturer!
05	Software sequence	Put into flight mode, wait 10 sec. Put into driving mode! Malfunction still present? Yes: Contact the manufacturer!
06	Wrong command	Put into flight mode, wait 10 sec. Put into driving mode! Malfunction still present? Yes: Contact the manufacturer!

Malfunction, powered wheels

Code	Malfunction	Troubleshooting
01	Brake	Contact manufacturer!
02	Motorcontroller	Contact manufacturer!
03	Electronic	Contact manufacturer!
04	Parameter data	Contact the manufacturer or authorized personnel!
05	Software sequence	Turn off the SOLO, wait for 10 sec., turn on the SOLO. Malfunction still present? Yes: Contact the manufacturer!
06	Wrong command	Turn off the SOLO, wait for 10 sec., turn on the SOLO. Malfunction still present? Yes: Contact the manufacturer!

Malfunction, control unit

Code	Malfunction	Troubleshooting
01	Keypad	Contact the manufacturer or authorized personnel!
02	Joystick	Contact the manufacturer or authorized personnel!
03	Serial number	Contact the manufacturer or authorized personnel!
04	Parameter data	Turn off the SOLO, wait for 10 sec., turn on the SOLO. Malfunction still present? Yes: Contact the manufacturer!
05	Software sequence	Turn off the SOLO, wait for 10 sec., turn on the SOLO. Malfunction still present? Yes: Contact the manufacturer!
06	Wrong command	Turn off the SOLO, wait for 10 sec., turn on the SOLO. Malfunction still present? Yes: Contact the manufacturer!
07	No contact to the monitoring of the getting up function	Contact the manufacturer or authorized personnel!

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7.3 Troubleshooting

Problem	Check and potential solution
SOLO cannot be turned on	Is the interface connected? No? Connect it.
	Is the helix cable plugged in? No? Plug it in.
	Is the battery pack empty? Yes? Charge it.
	Is the battery pack in driving mode? No? Go into driving mode.
	Problem still there? Yes? Contact the manufacturer or authorized personnel!
SOLO does not drive	Is the charger still plugged in? Yes? Unplug it.
	Is the SOLO still turned on? No? Turn it on.
	Is the immobiliser activated? Yes? Deactivate immobiliser.
	Are powered wheels locked? No? Lock them.
	Are powered wheels engaged? No? Engage them.
	Problem still there? Yes? Contact the manufacturer or authorized personnel!
Battery pack doesn't charge	Is the charger connected? No? Connect it properly.
	Problem still there? Yes? Contact the manufacturer or authorized personnel!

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8 Warranty and liability

8.1 Warranty

AAT Alber Antriebstechnik GmbH grants a warranty of two years for the SOLO including all its accessory parts (with the exception of the battery pack). The warranty begins at the date of purchase and covers failure of material and errors in manufacturing.

For the battery pack, always assuming correct maintenance AAT Alber Antriebstechnik GmbH grants a warranty of one year from the date of purchase.

The warranty becomes void concerning defects or damage which are the result of the following:

- » damage due to excessive strain.
- » damage due to improper handling.
- » violent damage.
- » natural wear and tear (wheelchair wheels).
- » improper and insufficient maintenance.
- » unauthorized constructional changes or inappropriate use of the SOLO or its accessories.
- » Using the SOLO exceeding the approved total weight.
- » improper charging of the battery pack.

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8.2 Liability

The manufacturer AAT Alber Antriebstechnik GmbH, is neither liable nor responsible for potential damage of the SOLO in case of:

- » improper handling of the SOLO.
- » repairs, fittings or other works performed on the SOLO by unauthorized personnel.
- » operation not in accordance with the user's manual.
- » extraneous parts added or connected to the SOLO.
- » parts removed from the SOLO.
- » operation exceeding the approved total weight.
- » no regular biannual maintenance check performed by AAT Alber Antriebstechnik GmbH or its authorized personnel.
- » operation of the SOLO with insufficiently charged batteries.

Important safety advice for the US market!

Warning: Radio wave sources may affect powered devices' control units!

Radio wave sources, such as radio stations, TV stations, amateur radio (HAM) transmitters, two-way radios, and cellular phones, can affect your SOLO.

Following the warnings listed below should reduce the chance of unintended brake release or movement, which could result in serious injury.

- » Do not turn ON hand-held personal communication devices, such as citizens band (CB) radios and cellular phones, while the SOLO is turned ON.
- » Be aware of nearby transmitters, such as radio or TV stations, and try to avoid coming close to them.
- » If unintended movement or brake release occurs, turn the powered SOLO OFF as soon as it is safe.
- » Be aware that adding accessories or components, or modifying the SOLO, may make it more susceptible to interference from radio wave sources (Note: There is no easy way to evaluate their effect on the SOLO's overall immunity).
- » Report all incidents of unintended movement or brake release to the SOLO manufacturer, and note whether there is a radio wave source nearby.

Important EMI-Information:

- » 20 Volts per meter (V/m) is a generally achievable and useful immunity level against interference from radio wave sources (as of May 1994) (the higher the level, the greater the protection).
- » This device has an immunity level of 20 V/m with no accessories connected to the device.
- » Nevertheless, always check whether the device's electronic functions work properly before each use. This is particularly important, if there is an aerial mast nearby.
- » Note that the SOLO may interfere with other electronic devices' sensitive electro-magnetic fields (e.g. anti-theft systems in department stores). This may also happen, if you connect other devices.
- » Also make sure that the SOLO is not operated in close proximity to medical devices with a high potential for danger and/or life-supporting functions as well as diagnostic devices.

D

GB

Notizen:

Notes:

81

D

GB

Notizen:

Notes:

82

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
D

GB

Notizen:

Notes:

83

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